WelchAllyn®

**Ophthalmoscopes,
Otosopes and
No. 20000
Otoscope/Throat
Illuminator**

Part No. 200066-10 Rev. D

For guaranteed
performance,
use only genuine
Welch Allyn Lamps

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**Operating and Cleaning
Instructions, Lamp
Replacement Chart**

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Italiano19

Operating Instructions for No. 20000 Otoscope/Throat Illuminator


To attach Otoscope/throat illuminator to power source:

Align cutouts in the Otoscope/Throat Illuminator base with lugs on power source. Push in slightly and turn 90 degrees. The Otoscope/Throat Illuminator is now ready to use. To remove, reverse this procedure. The

Otoscope/Throat Illuminator fits all 3.5 v. Welch Allyn power sources.

To use throat Illuminator:

1. Grasp the otoscope section with the left hand.

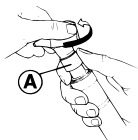2. With the right hand, grasp the throat illuminator's collar at A (under the silver ring).

3. While holding the throat illuminator collar stationary, twist the otoscope section counterclockwise at A until the otoscope section becomes loose.

4. Slide the right hand down to the power handle, grasp it and turn it clockwise while keeping the otoscope section stationary, until otoscope section is removed. The throat Illuminator is now ready for use.

To replace otoscope section, reverse the above steps.

Note: It is important to note that while the throat illuminator section provides an excellent source of general illumination, more than momentary direct skin contact should be avoided due to possible heat build-up.

Lamp Replacement Instructions for Welch Allyn Otoscoopes

Fiber Optic Otoscopes

Conventional Otoscopes

Lamp Replacement Instructions for Welch Allyn Ophthalmoscopes

Lamp Replacement Caution:

Halogen lamps are pressurized to provide maximum efficiency and illumination. Mistreatment may cause shattering. Protect lamp surface against abrasions and scratches. Use protective eyeglasses when handling. Be sure power is off when replacing lamp. Dispose of lamp with care.


CAUTION: Lamps may be hot. Lamp should be allowed to cool before removal.

Cleaning Recommendation

Repairs and cleaning (with the exception of lamp replacement) beyond what is specified in the general cleaning instructions section should be performed at the factory. Do not apply alcohol, chemicals or water to the mirror or lenses.


Ophthalmoscopes

No. 11470, 11475 & No. 11710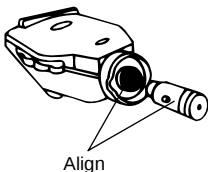

Ophthalmoscopes

No. 11720, 11730 & No. 11735

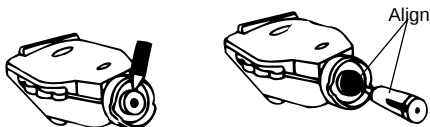General Cleaning Instructions

Note: Excess solution entering the optical assembly could damage internal components. Use caution to ensure cloth is not saturated with solution.

The otoscope and ophthalmoscope instrument heads may be cleaned with a dry cloth. Windows may be cleaned with a cotton swab dipped in alcohol or lens paper.

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Skaneateles Falls, NY 13153-0220 U.S.A.
Printed in U.S.A.